



INSTRUCTIONS FOR USE

Disposable Sterile Syringe

Product Name: Disposable Sterile Syringe

Product Model: 1ml,2ml,3ml,5ml,10ml,20ml,30ml,50ml

Production company: Anhui JN Medical Device Co.Ltd.

Registered address: Caicun Town Jing County Xuancheng City,Anhui,China.

Manufacturer Address: Caicun Town Jing County Xuancheng City,Anhui,China.

sale service unit: Caicun Town Jing County Xuancheng City,Anhui,China.

Product standard: ISO7886-1

※**Product consist:** plunger, barrel, piston, nozzle, luer-adapted

※**Declaration:** Free of silicone,Latex-free

※**Instructions for use:**

- 1、 Wash your hands before use;
- 2、 Wear a pair examination glove;
- 3、 Open the small package;
- 4、 Remove the Disposable sterile needle syringe;
- 5、 Remove the protective sleeve;
- 6、 Push the needle through the rubber stopper of the drug soup vial;
- 7、 Choose an injection sit, make sure the skin clean, pinch up a large area of skin, push the needle into the skin going straight in at a 90 degree angle, make sure the needle is all the way in;
- 8、 Push the plunger all the way down to inject the drug soup into your body

※**Contradictions in use**








- 1 、 It must be used before the expiry date, and products can not be used hat are past their expiry date
- 2 、 Check single package before use. Don't the product And If there is any other substance in it, forbidden to use.
- 3、 Only for single-use, and immediately after unsealed.
- 4、 Don't use it when packages are broken.
- 5 、 Don't use it when any other abnormal conditions appeared



※**Note:**

- 1、 Please read the instructions carefully before use, use in the warranty period.
- 2、 After use, according to the hospital or the requirements of the environmental protection departments for processing.
- 3、 This product is limited to single use packaging; if damaged or wet, do not use.
- 4、 For a qualified physician, nurse operation.

[Product labeling and packaging explanation]

			
Read the instructions	Do not use again	Sterilized by ethylene oxide	Do not use the packing
			
Afraid of the sun	Moisture-proof	Latex-free	

※Storage conditions: Packing storage in relative humidity of not more than 80%, cool and dry, ventilated, non corrosive gas chamber.

※Sterilization failure period: Five years


DATA SHEET

Specifications: 1ml Luer Lock

CONTENT

1. Data sheet
2. Brief introduction
3. Principle & intended use
4. Applicable standards

1.Data sheet

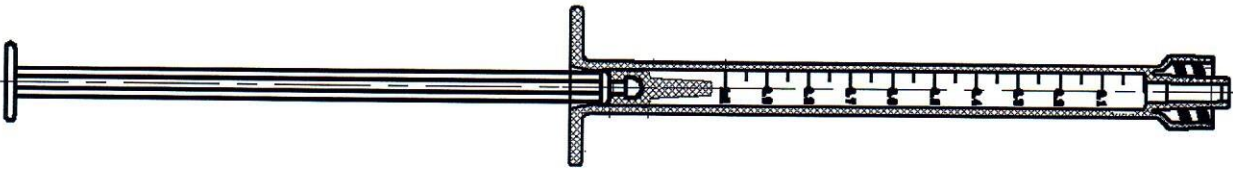				
Disposable sterile syringe		Anhui JN Medical Device Co.Ltd. 安徽江南医疗器械股份有限公司		
S-14,S-01,S-15,S-02,S-18,S-05				
NO.	Part name	QTY	Material	Note
1	Plunger	1	PP	Transparent
2	Barrel	1	PP	Transparent
3	Piston	1	Synthetic rubber	Black
4	Nozzle	1	PP	Transparent
5	Luer-Adapted	1	PP	Transparent

1. Brief introduction

The disposable syringe is for human body skin, hypoderm, muscle, vein and such injection usage.

The product is mainly made up of the barrel, plunger, piston. The products is sterile, no pyrogen, and all the indexes are up to the standard.

2. Principle & intended use

Disposable Sterile Syringe is a sterile, single-use, and disposable syringe which is intended to provide a safe and reliable method for intramuscular and subcutaneous injection of medication into patient.

3. Applicable standard:

ISO 7886-1-2017, With 0.1ml Graduation, Sterile.



Certificate

The Certification Body of
TÜV Rheinland LGA Products GmbH

hereby certifies that the organization
Anhui JN Medical Device Co., Ltd.
Caicun Town, Jing County
Xuancheng City
242525 Anhui
China

has established and applies a quality management system for medical devices
for the following scope:

Manufacture and Distribution of Medical Devices
(see attachment for products included)

Proof has been furnished that the requirements specified in

EN ISO 13485:2016

are fulfilled. The quality management system is subject to yearly surveillance.

Effective Date: 2019-06-19
Certificate Registration No.: SX 60140170 0001
An audit was performed. Report No.: 15062350 009
This Certificate is valid until: 2021-10-18

Certification Body



Date 2019-06-19



TÜV Rheinland LGA Products GmbH - Tillystraße 2 - 90431 Nürnberg
Tel.: +49 221 806-1371 Fax: +49 221 806-3935 e-mail: cert-validity@de.tuv.com http://www.tuv.com/safety

TÜV Rheinland
LGA Products GmbH
Tillystraße 2, 90431 Nürnberg

**Attachment to
Certificate**

Registration No.: SX 60140170 0001
Report No.: 15062350 009

Organization: Anhui JN Medical Device Co., Ltd.
Caicun Town, Jing County
Xuancheng City
242525 Anhui
China

Scope:

Products:

- Infusion Sets for Single Use
- Scalp Vein Sets for Single Use
- Transfusion Sets for Single Use
- Sterile Hypodermic Syringes for Single Use
- Sterile Hypodermic Needles for Single Use
- Infusion Sets with Precision Filters for Single Use
- Irrigation Syringes
- Disposable Transfer Sets
- Disposable Insulin Syringes

Certification Body



Date: 2019-06-19

Fuxiu Sheng



EC Certificate
Directive 93/42/EEC Annex V
Production Quality Assurance
Medical Devices

Registration No.: DD 60140171 0001

Report No.: 15062350 009

Manufacturer: Anhui JN Medical Device Co., Ltd.
Caicun Town, Jing County
Xuancheng City
242525 Anhui
China

Products: Medical Devices
(see attachment for products included)

Replaces Approval, Registration No.: DD 60135530 0001

Expiry Date: 2023-10-18

The Notified Body hereby declares that the requirements of Annex V of the directive 93/42/EEC have been met for the listed products. The above named manufacturer has established and applies a quality assurance system, which is subject to periodic surveillance, defined by Annex V, section 4 of the aforementioned directive. For placing on the market of class IIb and class III devices covered by this certificate an EC type-examination certificate according to Annex III is required.

Effective Date: 2019-06-19

Date: 2019-06-19



TÜV Rheinland LGA Products GmbH - Tillystraße 2 - 90431 Nürnberg
TÜV Rheinland LGA Products GmbH is a Notified Body according to Directive 93/42/EEC concerning medical devices with the identification number 0197.

TÜV Rheinland
LGA Products GmbH
Tillystraße 2, 90431 Nürnberg

**Attachment to
Certificate**

Registration No.: DD 60140171 0001
Report No.: 15062350 009

Manufacturer: Anhui JN Medical Device Co., Ltd.
Caicun Town, Jing County
Xuancheng City
242525 Anhui
China

Products:

Infusion Sets for Single Use, Scalp Vein Sets
for Single Use, Transfusion Sets for Single Use,
Sterile Hypodermic Syringes for Single Use,
Sterile Hypodermic Needles for Single Use, Infusion
Sets with Precision Filters for Single Use, Disposable
Insulin Syringes;

Aspects of manufacture concerned with securing and
maintaining sterile conditions:

Irrigation Syringes, Disposable Transfer Sets

Date: 2019-06-19

Notified Body

Fuxiu Sheng




DISPOSABLE SYRINGE

WITHOUT NEEDLE
STERILIZED BY E.O.GAS

Non-bacteria
Pyrogen free
Non-toxic

Set is ready for use
Single use only

Valid for five years

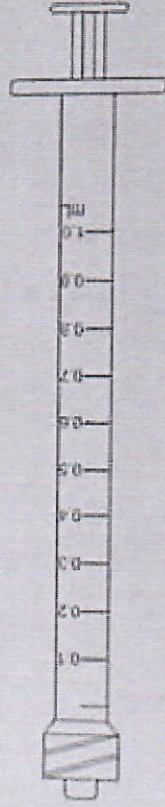

1 ML LUER LOCK

CE 0197    ISO 13485

STERILE EO

MANUFACTURER:



JN MEDICAL

ANHUI JN MEDICAL DEVICE CO., LTD.
CAICUN TOWN, JING COUNTY, XUANCHENG CITY, ANHUI, CHINA

Imported/Distributed by: PPE2020 srl. Via Alfredo Casella 16, 00199 Roma, Italy.
EG REF: Prolinx GmbH, Brehmstr. 56, DE-40239 Duesseidorf, Germany

100 pcs

MANUFACTURER
ANHUI JIN MEDICAL DEVICE CO., LTD.

STERILE EO 1 ml/cc ISO13485

DISPOSABLE SYRINGE
MANUFACTURER:
ANHUI JIN MEDICAL DEVICE CO., LTD.
CASAS TOWN AND COUNTY JIANGCHENG CITY ANHUIC
INGREDIENTS: Polystyrene, PPS200 etc. Via Ahmed Hassan St. 81199 Mainz, DE

WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.
Produced in Germany

LOT: 20210101
EXP: 2025-12
Luer Lock

STERILE EO 1 ml/cc ISO13485

DISPOSABLE SYRINGE
MANUFACTURER:
ANHUI JIN MEDICAL DEVICE CO., LTD.
CASAS TOWN AND COUNTY JIANGCHENG CITY ANHUIC
INGREDIENTS: Polystyrene, PPS200 etc. Via Ahmed Hassan St. 81199 Mainz, DE

WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.
Produced in Germany

LOT: 20210101
EXP: 2025-12
Luer Lock

STERILE EO 1 ml/cc ISO13485

DISPOSABLE SYRINGE
MANUFACTURER:
ANHUI JIN MEDICAL DEVICE CO., LTD.
CASAS TOWN AND COUNTY JIANGCHENG CITY ANHUIC
INGREDIENTS: Polystyrene, PPS200 etc. Via Ahmed Hassan St. 81199 Mainz, DE

WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.
Produced in Germany

LOT: 20210101
EXP: 2025-12
Luer Lock


STERILE EO 1 ml/cc ISO13485

DISPOSABLE SYRINGE

WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.
Produced in Germany

LOT: 20210101
EXP: 2025-12
Luer Lock

LUER LOCK
LOT : 20210101
: 2021-01
: 2025-12

 1 ml/cc **STERILE EO**


REF S-14 WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC, NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.

 Prolinx GmbH, Brahmstr. 56, DE-40239 Duesseldorf, Germany


DISPOSAB

MANUFACTURER:
ANHUI JN MEDICAL
CAICUN TOWN, JING COUNTY, XUANCHENG
Imported/Distributed by: PPE2020

LUER LOCK
LOT : 20210101
: 2021-01
: 2025-12

 1 ml/cc **STERILE EO**


REF S-14 WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC, NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.

 Prolinx GmbH, Brahmstr. 56, DE-40239 Duesseldorf, Germany

DISPOSAB

MANUFACTURER:
ANHUI JN MEDICAL
CAICUN TOWN, JING COUNTY, XUANCHENG
Imported/Distributed by: PPE2020

LUER LOCK
LOT : 20210101
: 2021-01
: 2025-12


 1 ml/cc **STERILE EO**

REF S-14 WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC, NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.



 Prolinx GmbH, Brahmstr. 56, DE-40239 Duesseldorf, Germany

DISPOSAB

MANUFACTURER:
ANHUI JN MEDICAL
CAICUN TOWN, JING COUNTY, XUANCHENG
Imported/Distributed by: PPE2020

 1 ml/cc **STERILE EO**

DISPOSAB

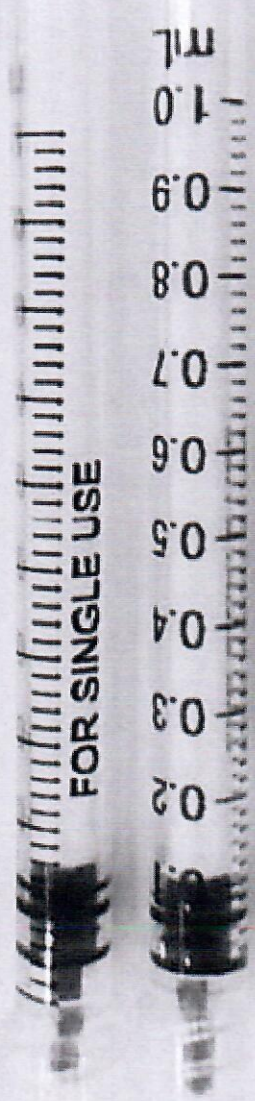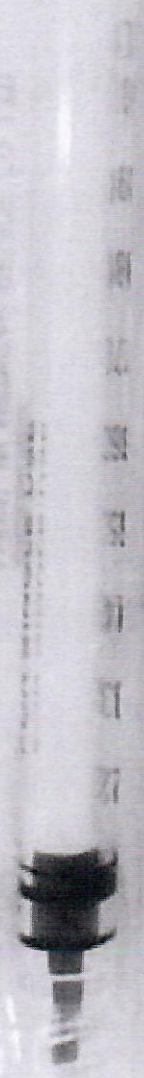

FOR SINGLE USE



1 ml/cc

STERILE EO



CE 0197

ISO13485

REF S-14 WITHOUT NEEDLE
STERILIZED BY E.O.GAS
NON-TOXIC, NON-PYROGENIC
SINGLE USE ONLY
DO NOT USE IF PACKAGE IS DAMAGED.

DISPOSABLE SYRINGE

MANUFACTURER:

ANHUI JN MEDICAL DEVICE CO.,LTD.

CACUN TOWNSHIP COUNTY JUANCHENG CITY ANHUI CHINA

Imported/Distributed by: PFE3020 srl Via Alfredo Casella 16, 00199 Roma, Italy



LOCK 20210101
2021-01
2025-12